

# CHIRAFLEX

## Mandrin - Luer Lock

### Instruction for Use STYLET (Obturator) for Single Use



Non-Toxic  
Non-Pyrogenic

- Description:** Obturator for single use. Sterile, Non-toxic, Non-pyrogenic.  
**Components:** Obturator.  
**Application scope:** To keep Intravascular Cannulae closed for longer duration.  
**Prohibition:** Not to be used in patients with known hypersensitivity to any of the materials used (Polypropylene).



1. Before usage, confirm the sterility validity, packing completeness, the lot No. and expiry date are in order;
2. The products should be used immediately after envelope opened, discard after use;
3. To be used by only a qualified doctor or paramedic;
4. Only for single use. Discard the set after single use;
5. Re-sterilization or re-use could cause infection or cross-interactions.



1. Select proper type of obturator as per the size of Intravascular cannulae;
2. Inspect package for integrity and expiry and then remove the obturator from the package;
3. Remove the lure cap from the intravascular catheter carefully and insert the obturator throughout the catheter.
4. Close fit the obturator into Intravascular Catheter;
5. Remove the obturator when the drug or medicine is to be administrated.



Attention



See instruction for use



Serial number



Batch Number



Date of Manufacturing



Use by / Expiry date



Sterilised by Ethylene Oxide



Manufactured by



For single use only



Do not Resterilize



Do not use if unit package is damaged.



Pyrogen free



Keep away from sunlight.



Temperature Limitation.



Keep away from Rain.

 **CHIRANA T. Injecta, a.s.**  
 Nám. Dr. Schweitzera 194  
 916 01 Stará Turá  
 Slovak Republic  
 Tel.: +421/32/7709 991

[www.t-injecta.com](http://www.t-injecta.com)